DE

Kontraindikationen

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EN

Contraindications

Basically, general medical as well as local, absolute and relative contraindications for dental surgical measures must be observed.

IMPORTANT: Attention must be paid to protecting anatomical structures (safety clearance at least 2 mm) as well as the gradient of the adjacent teeth / tooth roots (risk of damage, infection / dehiscences).

ATTENTION: The instruments are not intended for final implant site preparation. The diameter as well as the fine preparation of the implant site must be produced with the instruments of the planned implant system according to the manufacturer's instructions.

ATTENTION: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

General instructions

Please follow general application and safety instructions for MEISINGER products in the medical area and also the advice for processing (cleaning, disinfection and sterilisation) of medical devices from Hager & Meisinger GmbH.

Hager & Meisinger GmbH
Hansemannstr. 10
41468 Neuss • Germany
Tel. +49 2131 2012-0
Fax: +49 2131 2012-222
Email: info@meisinger.de
Internet: www.meisinger.de

Meisinger USA, L.L.C.
10150 E. Easter Avenue
Centennial, Colorado 80112 • USA
Phone: +1 (303) 268-5400
Fax: +1 (303) 268-5407
Email: info@meisingerusa.com
Internet: www.meisingerusa.com

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Inhalt / Content / Contenido / Contenu / Contenuto

Drill-Stop-Control:

Diagram and table for Drill-Stop-Control. Includes illustrations of initial bar with stop and twist drills, and a table with columns for Fig. (187RF, TDS15), Shank, Size, Length, and Opt. speed.

Diagram and table for Stop sleeves. Includes illustrations of stop sleeves and a table with columns for Fig. (CL031-CL036) and Length.

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Drill-Stop-Control BLT:

Diagram and table for Drill-Stop-Control BLT. Includes illustrations of initial bar with stop and twist drills with BLT, and a table with columns for Fig. (187RF, TDS0T), Shank, Size, Length, and Opt. speed.

Diagram and table for Stop sleeves BLT. Includes illustrations of stop sleeves and a table with columns for Fig. (CL066-CL061) and Length.

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Drill-Stop-Control myplant two:

Diagram and table for Drill-Stop-Control myplant two. Includes illustrations of initial bar with stop and twist drills, and a table with columns for Fig. (187RF, TDS18), Shank, Size, Length, and Opt. speed.

Diagram and table for Stop sleeves myplant two. Includes illustrations of stop sleeves and a table with columns for Fig. (CL050-CL057) and Length.

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