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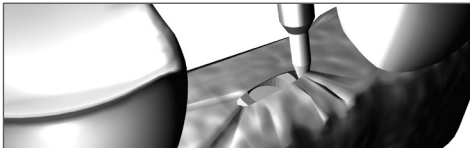
Application and safety instructions for the MEISINGER

Gingiva Trimmer

The tissue saving alternative to the scalpel

Art.-No.: GT48L, GT135

The MEISINGER gingiva trimmers are intended for use in dental mucosal surgery. They have been specially developed for the versatile and gentle treatment of gingival tissue. The working part consists of particularly high-quality and stable zirconia. The main application focus of the MEISINGER gingiva trimmers is the safe and gentle removal of unwanted granulation tissue and hyperplastic gingiva (papillectomy) - even in hard-to-reach areas. Furthermore, the trimmers are used in the enlargement of the sulcus after crown preparation, for the best possible exposition of the preparation margin and the creation of an exact impression. The gingiva trimmers are also particularly suitable for the tissue saving exposure of covered healed implants and for use in periodontal treatment. The application takes place without cooling, so that the resulting rotational energy can be used for tissue modulation. Thanks to heat coagulation, bleeding tendency is reduced.

Indications for Use

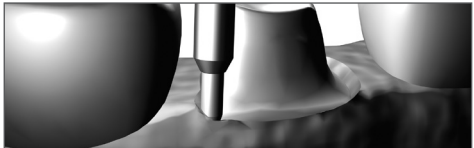
The gingiva trimmers are indicated for the opening of the sulcus to allow the most accurate impression possible, for exposing deep-seated cavities (dental cavities) and retained teeth as well as for removing unwanted granulation tissue and hyperplastic gingiva (papillectomy). In addition, the gingiva trimmers are used to expose intraosseous implants. The gingiva trimmers are also ideally suited for use in periodontal treatment.

Utilization


The application of the products must be carried out without any cooling at a speed of 300,000 - 450,000 rpm, whereby a modeling of the soft tissue by heat coagulation with reduced bleeding tendency is possible. This achieves a clean, almost bloodless working area with low heat generation.

Application possibilities


Opening of the sulcus after crown preparation

Removing of hyperplastic gingiva

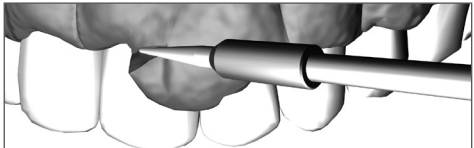Exposure of intraosseous implants

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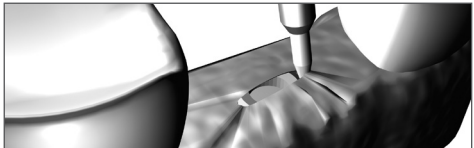Contraindications

Basically, general medical as well as local, absolute and relative contraindications for dental surgical procedures must be considered.

CAUTION: Leverage, which may cause tilting, should be avoided.

CAUTION: A lower applied speed than 300,000 - 450,000 rpm can lead to injuries and should therefore be avoided. The instruments must be used without cooling, otherwise coagulation of the capillaries will be prevented.

NOTE: With careful use and proper processing, the gingiva trimmers can be used multiple times. The exact service life of the products depends on the correctness and intensity of applications and subsequent processing. Care should be taken to ensure that no sharp instruments are used for cleaning the ceramic tip and that the instruments are processed separately.

ATTENTION: Any serious-incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

General instructions

Please follow general application and safety instructions for MEISINGER products in the medical area and also the instructions for processing (cleaning, disinfection and sterilisation) of medical devices from Hager & Meisinger GmbH.

Hager & Meisinger GmbH
Hansemannstr. 10
41468 Neuss • Germany
Tel. +49 2131 2012-0
Fax: +49 2131 2012-222
Email: info@meisinger.de
Internet: www.meisinger.de

Meisinger USA, L.L.C.
10150 E. Easter Avenue
Centennial, Colorado 80112 • USA
Phone: +1 (303) 268-5400
Fax: +1 (303) 268-5407
Email: info@meisingerusa.com
Internet: www.meisingerusa.com

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