MEISINGER

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Instructions For Use MEISINGER Easy-Clean Gum Scissors

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1. For dismantling, open the scissors in the right angle (90°).

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2. Lift the upper part of the scissors (ca. 45°) and separate both parts from each other. By appropriate handling, the scissors parts can be separated without great effort - otherwise risks of damage / breakage of the screw cannot be excluded.

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3. After processing has been effected, both scissors parts are put together again in reversed order.

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4. Both scissors parts are marked inside with a identification number (equal for both sides), so that the matching parts can be assembled each time again.

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Note:

Must be checked for fault-free operation (fractures, cracks, malfunctions etc.) and be cleaned, disinfected and sterilized before each use. New instruments are contaminated with oils and fats from the manufacturing process and must therefore be cleaned thoroughly before the first sterilisation with a suitable cleaning agent.

Further instructions for processing can be found at www.meisinger.de

ATTENTION: Any serious-incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Hager & Meisinger GmbH