


Periimplantitis-Kit

Long Tungsten Carbide instruments for ideal peri-implantitis treatment

The new MEISINGER Periimplantitis Kit contains eight perfectly matched tungsten carbide finishing burs in egg and flame shape for the ideal peri-implantitis treatment. There are both shapes in two sizes and each in standard and extra fine toothings. The shapes, sizes and toothings of the working parts are perfectly suited for the intraoral treatment of titanium. All finishers have an extra-long FG shank (total length 32 mm) which allows particularly the treatment of difficult to access, deep areas. For the peri-implantitis treatment the finishers are guided around the exposed implant anti-clockwise, so the implant surface is smoothed.

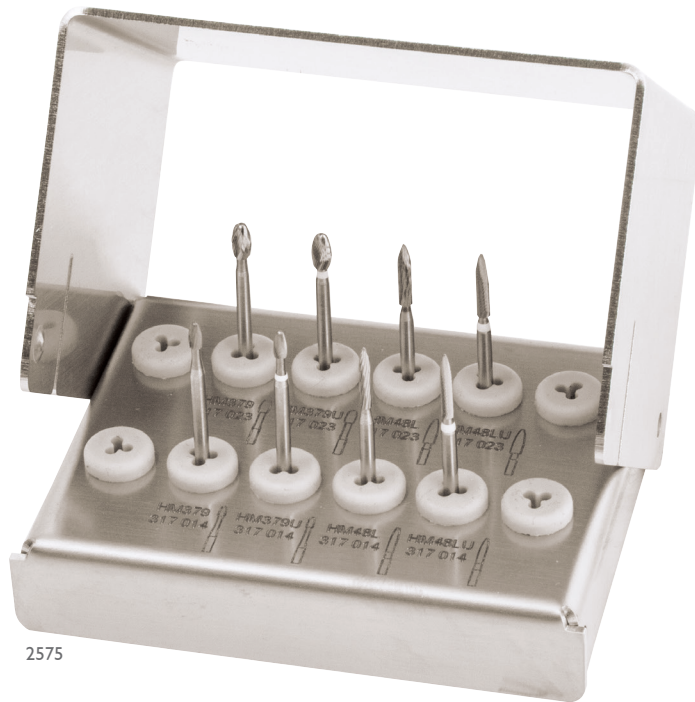

2575

Made in
Germany

At a glance






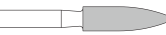




- Perfectly matched tungsten carbide finishing burs for the effective, intraoral treatment of titanium
- XXL FG shanks facilitate the treatment of difficult to access implants
- Different shapes and sizes for every individual patient case
- Clean and smooth titanium surfaces

Our sales force would be pleased to
answer your questions: +49 2131 2012-303

Meisinger
since
1888

Periimplantitis-Kit

Fax for ordering +49 2131 2012-222

	Figur	Shank	Size	recommended / max. revolutions	Pieces / Kit	Amount 
	2575	-	-	-	-	
	HM379	XXLFG (317)	014	40.000 min ⁻¹ / 300.000 min ⁻¹	1	
	HM379	XXLFG (317)	023	40.000 min ⁻¹ / 190.000 min ⁻¹	1	
	HM48L	XXLFG (317)	014	40.000 min ⁻¹ / 120.000 min ⁻¹	1	
	HM48L	XXLFG (317)	023	40.000 min ⁻¹ / 120.000 min ⁻¹	1	
	HM379U	XXLFG (317)	014	40.000 min ⁻¹ / 300.000 min ⁻¹	1	
	HM379U	XXLFG (317)	023	40.000 min ⁻¹ / 190.000 min ⁻¹	1	
	HM48LU	XXLFG (317)	014	40.000 min ⁻¹ / 120.000 min ⁻¹	1	
	HM48LU	XXLFG (317)	023	40.000 min ⁻¹ / 120.000 min ⁻¹	1	

CE 0044

Available from the dental trade.

Your trade depot:

Your delivery address:

Hager & Meisinger GmbH

Hansemannstr. 10 • 41468 Neuss • Germany
 Phone: +49 2131 2012-0 • Fax: +49 2131 2012-222
 info@meisinger.de • www.meisinger.de

Meisinger USA. L.L.C.

10150 E. Easter Avenue • Centennial, Colorado 80112 • USA
 Phone: +1 (303) 268-5400 • Fax: +1 (303) 268-5407
 Toll free: +1 (866) 634-7464
 info@meisingerusa.com • www.meisingerusa.com

85FL118EN - 0317